# PEARL Nitrile Examination Gloves

PERFECT FIT • FINGER TEXTURED
• POWDER FREE • LATEX FREE • AMBIDEXTROUS • NON-STERILE

vrs:78-2507

<b>CE</b> <b>MDR (EU) 2017/745 (Class I Medical Device)</b> <b>2834 (PPER (EU) 2016/425) (Cat. III, Type C)</b>	
<small>Notified Body responsible for Certification and ongoing conformity: <b>CCQS Certification Services Limited</b> Block 1 Blanchardstown Corporate Park, Ballycoolin Road, Blanchardstown, Dublin, 15 D15 AKK1, Ireland (NB 2834)</small>	
<small>Polo MB Oisterwijk BV Lazarkweg 5 5061 JR Oisterwijk, The Netherlands www.poldomb.eu • info@poldomb.eu</small>	<small>UNIGLOVES Agt- &amp; Klinikbedarf Handelsgesellschaft mbH Camp-Spich-Strasse 71, D-53842 Troisdorf, Germany</small>
<b>MD</b>	EN 455-1 EN 455-2 EN 455-3 EN 455-4

**en** Nitrile Premium examination gloves powder-free

**2016/425 #B & #C2; CCQS Certification Services Ltd, #1 Blanchardstown Corporate Park, D15 AKK1, IRL (2834)** You are advised to retain this packaging for reference. Contains accelerators which may cause allergic reactions. This information does not reflect the actual duration of protection in the workplace and the differentiation between mixtures and pure chemicals. It is recommended to check that the gloves are suitable for the intended use because the conditions at the workplace may differ from the type test depending on temperature, abrasion and degradation. When used, protective gloves may provide less resistance to the dangerous chemical due to changes in physical properties. Movements, snagging, rubbing, degradation caused by the chemical contact etc. may reduce the actual use time significantly. For corrosive chemicals, degradation can be the most important factor to consider in selection of chemical resistant gloves. Before usage, inspect the gloves for any defect or imperfections. Storage: store in dry conditions away from direct sunlight and heat. EN ISO 374-4:2019 Degradation levels indicate the change in puncture resistance of the gloves after exposure to the challenge chemical. The penetration resistance has been assessed under laboratory conditions and relates only to the tested specimen. For single use only. The chemical resistance has been assessed under laboratory conditions from samples taken from the palm only (except in cases where the glove is equal to or over 400mm where the cuff is tested also) and relates only to the chemical tested. It can be different if the chemical is used in a mixture. Glove donning: Ensure the selection of appropriate glove size. Check for any physical damage or contamination prior to use. Discard if there is any damage or contaminant. Glove doffing: Remove the glove as soon as it wears out or is damaged. When contaminant is not removable or presents a potential hazard, it is advisable to remove left and right gloves alternately using the gloved hand so that gloves are removed without the contaminant touching the hand. Suitable for short term foot contact. The product can be transported at -10°C up to 50°C for up to 7 days.

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