

LEADING REGENERATION

Geistlich

# Geistlich Fibro-Gide®

*The alternative connective tissue graft.*  
For you and your patient.

NEW  
SIZES



# Expanding our soft-tissue portfolio – For you and your patients

Geistlich Fibro-Gide® is now available in 4 different sizes.  
Same indications, more treatment options.



6 mm thickness: 15 x 20 and 20 x 40

**3 mm thickness: 15 x 20 and 20 x 40**

NEW

- > Convenient: Available now in a reduced thickness of 3 mm
- > Easy handling: Easy to use, wet, cut and suture<sup>1</sup>
- > Palate free: No harvesting of patient's own tissue from the palate<sup>2</sup>
- > **Patient's choice: 66.7% of patients prefer Geistlich Fibro-Gide® over connective tissue graft treatment at 6 months<sup>3</sup>**
- > And more: With Geistlich Fibro-Gide® patient's pain perception and chair time are reduced<sup>4,5</sup>

## Treatment of Multiple Gingival Recession

Dr. Raffaele Cavalcanti | Bari, Italy



1 Baseline: multiple recession defects Miller Class I with keratinized tissue less than 3 mm on site 12 to 14.



2 3 mm Geistlich Fibro-Gide® is placed in the defect and absorbed blood immediately.



3 Positioning of Geistlich Fibro-Gide® and fixation with single sutures (7-0 PGA sutures).



4 Follow-up after 12 months: complete root coverage with Geistlich Fibro-Gide® is achieved.

# Reduced thickness – Same reliable product




„A Geistlich Fibro-Gide® thickness of 3 mm fits to the majority of single sites and thereby further reduces my time during surgery“


Prof. Daniel Thoma | Zurich, Switzerland

## Soft-tissue Thickening Around Dental Implants and Under Pontics

Prof. Jan Cosyn | Gent, Belgium

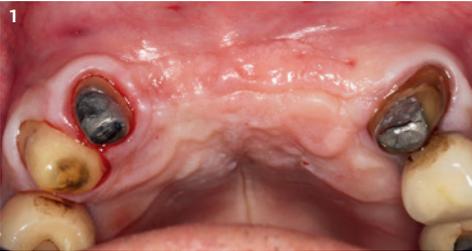

1 Pre-operative view showing lack of buccal convexity.

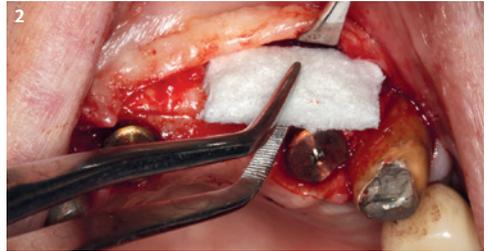

2 After full-thickness mucoperiosteal flap elevation, implants were installed in position 11 and 22 and two Geistlich Fibro-Gide® 3 mm were placed.



3 Primary wound closure.



4 Emergence profile 3 months after soft-tissue condition with the provisional bridge.



More details about our distribution partners:  
[www.geistlich-dental.com](http://www.geistlich-dental.com)

## Manufacturer

Geistlich Pharma AG  
Business Unit Dental  
Bahnhofstrasse 40  
6110 Wolhusen, Switzerland  
Phone + 41 41 492 55 55  
Fax + 41 41 492 56 39  
[www.geistlich-dental.com](http://www.geistlich-dental.com)

Discover our soft-tissue solution

## Put palate free innovation into your practice

Want to find out more about palate free?



Are you interested in further treatment concepts?



### Geistlich Fibro-Gide®

6 mm thickness: 15 x 20 and 20 x 40 mm

**3 mm thickness: 15 x 20 and 20 x 40 mm**

NEW

### References

- 1 Data on file. Usability test Geistlich Fibro-Gide® 2020
- 2 Thoma DS et al.: J Clin Periodontol 2016; 43 (10), 874–85.
- 3 McGuire, M. et al.: 2021 Jul 21. doi: 10.1002/JPER.21–0131
- 4 Thoma DS et al.: J Clin Periodontol 2016; 43 (10), 874–85
- 5 De Angelis P, et al.: Int J Oral Maxillofac Surg. 2021;50(7):956–6313